

Material:

PP, PE, & SS

Instructions for use

- Choose the needle gauge (size of the needle) according to the drug that you want to administer, and attach the needle to the syringe.
- Determine the drug dosage. Before filling the syringe with the drug, fill it with an amount of air that is equal to the size of the drug dosage. For instance, if the drug dosage is 5mL, fill the syringe with 5mL of air. Once you've filled the syringe with the allotted amount of air, inject it into the drug vial (the container in which the drug comes.). Turn the vial upside-down and fill the syringe with the drug dosage. With the vial turned upside-down, the air inside will rise to the top, which helps you get an exact amount of medicine without bubbles in it.
- Choose an injection site. Try to find a site that hasn't been used recently, because overusing a site can be painful.
- Pinch the skin at the injection site and insert the needle at a 45-to-90-degree angle. Inject the drug slowly until the syringe is empty.
- Remove the needle and dispose of it. Wipe off the injection site and cover it with a small adhesive bandage.

INSTRUCTION FOR USE

Warning :

- To be use this product by a qualified doctor or a paramedic.
- Read instruction s before use.
- **MANUFACTURER DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.**
- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use.
- Improper transport and handling may cause structural and / or functional damage to device or packaging.
- The product is guaranteed non-toxic, sterile and Pyrogen free if the package is not opened or damage.
- Do not clean or re-sterilize.
- The product should be used immediately after opening the packaging.
- The device must not be re-used after its usage, it can be a potential biohazard.

Storage Condition :

- Do not expose to heat or direct sunlight.

 Disposafe Health and Life Care Limited

Plot No. 1 & 2, Phase-II, Sector-59, Ballabhgarh,
Faridabad-121004, Haryana, India
E-mail : info@disposafe.com,
Web : www.disposafe.com
Customer Care No. +91-129-4170201



Medical Device



Reference No.



Lot No./Batch No.



Manufacturer



Date Of Manufacturing



Use By / Expiry Date



Do Not Reuse



Do Not Resterilize



Non-Pyrogenic



Do Not Use If Package Is Damage
& Consult Instructions For Use
Caution



Consult Instructions For Use



Temperature Limit



Keep away from sunlight



Authorised Representative
In The European Community



Sterilized By Ethylene Oxide
Single Sterile Barrier System



CEpartner4U B.V., ESDOORNLAAN 13,
3951 DB MAARN, THE NETHERLANDS.
E-mail: office@cepartner4u.com
Mfg. Lie. No.: MFG/MD/2019/000106
AW/IFU_761, REV:03, DT 10-01-2024