

DISPOFIX SURGICAL DRESSING (I.V. FIXATOR)

MATERIALS USED:

1. COTTON FABRIC
2. HYPOALLERGENIC GLUE/ADHESIVE
3. RELEASE PAPER

INDICATIONS:

- Fixing The IV Cannula To Give Support So The Cannula Not Moving.

CONTRAINDICATIONS:

- Product should not be used in patients with known hypersensitivity to any of the materials used.

INSTRUCTION FOR USE:

- Slip Properly over the wings of IV Cannula and cover the Cannulation sight so the Viscose pad is placed exactly on the place where Cannulation insertion has made this perfection is very important for desired fixation and placement of I.V Cannula Fixator.
- Please ensure you are placing and covering both wings to provide proper support.
- Viscose Pad Must Cover the penetration point.

WARNINGS:

- The use of this product is restricted to a qualified doctor or a Paramedic
- Read Instruction before use.

MANUFACTURER DISCLAIMS ANY

Disposafe Health And Life Care Limited

Plot No. 1 & 2, Phase-II, Sector-59, Ballabhgarh,
Faridabad - 121004, Haryana, INDIA

E-MAIL : info@disposafe.com

Web : www.disposafe.com

Mfg.Lic.No.:MFG/MD/2019/000106

INSTRUCTIONS FOR USE

RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

- The product should not be reprocessed.
 - Visually inspect and carefully check the product and packaging before use. Improper transport and handling may cause structural and/or functional damage to device or packaging.
 - The product is guaranteed sterile & Non-pyrogenic if the package has not been opened or damaged.
 - Do not clean or resterilise. For single use only. Discard after use.
 - The product should be used immediately after opening the packaging.
- The device must not be re-used after its usage, it can be a potential biohazard.

STORAGE:

- Store in cool and dry place.
- Do not expose to heat or direct sunlight.

 0123



CEpartner4U B.V., ESDOORNLAAN 13,
3951 DB MAARN, THE NETHERLANDS
E-mail: office@cepartner4u.com

 MD

Medical Device

 REF

Reference No.

 LOT

Lot No./Batch No.



Manufacturer



Date Of Manufacturing



Use By / Expiry Date



Do Not Reuse



Do Not Resterilize



Non-Pyrogenic



Do Not Use If Package Is Damaged



Caution



Temperature Limit



Keep Away From Sunlight



Consult Instructions For Use



Sterilized by ethylene oxide
Single Sterile Barrier System



Authorised Representative
In The European Community

