

INSTRUCTION FOR USE

DISPOFLEX THREE WAY STOP COCK

Material:

Polypropylene, H.D.P.E, Polycarbonate

Indications:

- Infusion of I.V. Solution.
- Intermittent intravenous Drugs administration.

Contraindications:

- Product should not be used in patients with known hypersensitivity
- Not to be used for photo sensitive drugs.
- Not to be used for blood transfusion.
- Not to be used for high viscous fluids and lipids administration

Instructions for use:

- Check visually whether the package is intact.
- Open packaging and remove stopcock from sterile package.
- Connect one or two I.V. Infusion line as desired to the ports.
- Remove protector from male portion & hold it upright.
- Open clamps of I.V. Infusion line and displace air entirely from
- Stopcock by rotating tap to open position
- Rotate tap to close position.
- Connect it to veinpuncture device & secure it with rotating lock.
- Rotate the tap to open position and adjust the flow of infusion set.
- Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions of infections.
- Contains universal 6% taper that can be connected to any standard medical device.


Warning:

- To be use this product by a qualified doctor or a paramedic.
- Read instructions before use.
- The product should be used according to the instruction for use.
- MANUFACTURER DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.**
- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use.
- Improper transport and handling may cause structural and /or functional damage to device or packaging.
- The product is guaranteed non-toxic, sterile and Pyrogen free if the package is not opened or damaged.
- Do not clean or re-sterilize.
- The product should be used immediately after opening the packaging.
- The device must not be re-used after its usage, it can be a potential biohazard.


Storage Condition :

- Do not expose to heat or direct sunlight


 Medical Device


 Reference No.


 Lot No./Batch No.


 Manufacturer


 Date Of Manufacturing


 Use By / Expiry Date


 Sterilised By Ethylene Oxide
Single Sterile Barrier System


 Do Not Reuse

 Do Not Resterilize


 Non-Pyrogenic


 Do Not Use If Package Is Damaged

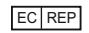
 Temperature Limit

 Caution

 Consult Instructions For Use

 Authorised Representative
In The European Community

 **Disosafe Health and Life Care Limited**
Plot No. 18 & 2, Phase-II, Sector- 59, Ballabhgarh,
Faridabad-121004, Haryana (INDIA)
E-mail: info@disosafe.com. Web: www.disosafe.com
CUSTOMER CARE NO. : +91-129-4170201

 CEpartner4U B.V
ESDOORNLAAN 13, 3951 DB MAARN,
THE NETHERLANDS
E-mail : office@cepartner4u.com

