

Instruction for use

- Close the roller clamp
- Remove the protective cover and insert the piercing device straight into the top of the bottle up to its full length
- Turn the bottle upside down and hang it at suitable height
- Squeeze & release the drip chamber until it is half filled
- Remove needle protector holding needle upright
- Open roller clamp and allow solution to pass until all air bubbles in the tube are removed
- Close the roller clamp
- Perform vein puncture and adjust the roller clamp to achieve desired flow rate.
- Post use this product may be a potential biohazard and must be disposed in compliance with state or country laws.

Warning:

- To check package is intact and about detached protective caps.

Caution :

- Do not use if package is open or damaged. Do not store in direct sunlight at extreme temperature or in high humidity. Discard after use. Do not use with blood or blood components. Do not re-use