


TEAR HERE

neb-kit™

Nebulizer Face Mask Kit With Star Lumen Tube

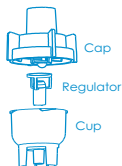

We Touch... Lives!

Instruction for Use:

- Unscrew the nebulizer cup from top and add medication as prescribed by physician. Re-attach cup to top. Medication may be added to Nebulizer without unscrewing the two parts.
- Insert Nebulizer into the mask connector.
- Connect the Multi Channel Tubing at the bottom of the Nebulizer cup, attached the other end of the tubing to compressed air source.

Cleaning Instructions:

- Dis-assemble Nebulizer cup from top and wash all parts (except tubing in water and non lemon type dishwashing liquid.
- Rinse thoroughly with warm water and air dry or hand dry with clean lint free cloth. When dry re-assemble medication kit by putting the regulator inside the cup and fix the cap on cup by pushing and then rotate the cap in clock wise direction. Store the kit for next use.

Disposafe Health And Lifecare Limited

Plot No.1 & 2, Phase-II, Sector-59, Ballabhgarh,
Faridabad-121004, Haryana INDIA

E-mail : info@disposafe.com

Web : www.disposafe.com **MD**


Customer Care No. 91-129-4170201

Mfg. Lic. No. MFG/MD/2019/000106

CE
0123

DEHP
FREE

1
Unit

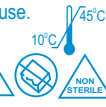

EC/REP

CEpartner4U BV., ESDOORNLAAN 13,
3951 DB MAARN, THE NETHERLANDS.
E-mail : office@cepartner4u.com

AW/IFU_150_REV:02

REF :

LOT :



SIZE :