



We Touch...Lives!

INSTRUCTION FOR USE STERILE DISPOSABLE SLIDE SAFETY SCALPEL

PRODUCT -

(Sterile Disposable Slide Safety Scalpel)

Surgical Blade fitted on a plastic handle with slide locking

MATERIALS USED

- Stainless Steel, ABS Plastic & PP

INDICATIONS :

- **Dissection Incision.**
- **Removal of extra tissue from operating area.**

CONTRAINDICATIONS:

- **Re-use of blades can work as carrier for communicable disease to patient and /or user.**
- **Device should not be used after expiry date.**

INSTRUCTIONS FOR USE :

- **Select proper type and size of Disposable scalpel as per the sight. always use bigger size for thick tissue.**
 - After selection clean the sight with proper disinfectant as per hospital protocol.
 - Inspect package of blade if intact then remove safety scalpel from package.
For making incision pl push botton turned to express the Blade full.
 - Incision has to place as per the desired length preferably measured and marked.
 - After Successful incision press the button and move the safety barrel backwards until hearing an click.
- The click confirms the activation of safety feature.

CAUTION / WARNINGS:

- Re-use may cause serious health deterioration.
- The product should be used only by a qualified surgeon or paramedic.
- Before use always check integrity of product and packing along with expiry date.
- Do not touch the sharp edges.
- Once the device is having sharp edges, keep it away from children's

For single use only.

- Use product immediately after opening the pack
- **DISPOSAFE DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.**
- DISPOSAFE do not hold any responsibility if device re-used or re-sterile.
- Sterility of product is guaranteed if packaging is not damaged.

STORAGE CONDITION: - Keep away from direct sun light. - Keep away from rain

- Storage temperature should be 10 to 40 degree celcius.
- Humidity of storage area should be RH 35% to 65 %.

DISPOSAL OF USED POUCH AND BLADES:

- The device facilitates Safety features to protect the user on the waste handlers from the sharp injury related contamination / injection.
- After using blade pieces and pouches shall be disposed off in compliance with the local regulatory requirements of handling medical use.

PRECAUTIONS:

- Always open the pouch from peel apart direction to avoid injury.
- Devices are extremely sharp, use carefully while handling.
- Care must be taken so that the pouch is not opened in an unsterile area otherwise the blade which has already been sterilized by EO Sterilization will become unsterile. Proper procedure must be used as applicable for handling any sterile product.
- Care must be taken during of device to avoid any contact or injury due to the sharp nature of the device.
- In case of changes in the performance of device for intended use replace the defective device by new to full fill the required application.

REF

Reference No



Caution/Warning (Read IFU before use)



Do Not Reuse

LOT

Lot Number / Batch Number



Date of Manufacturing



Use By / Expiry Date

STERILEEO

Sterilize using ethylene oxide



Manufactured by



Do not re-sterilize



Keep dry



Keep away from sunlight



Moisture limitation



Temperature Limitation



Do not use if package is damaged



Consult for IFU



Non-Pyrogenic

DISPOSAFE HEALTH AND LIFE CARE LIMITED

PLOT NO. 1 & 2, PHASE-II, SECTOR-59, BALLABGHARH,
FARIDABAD - 121004, HARYANA, INDIA
E-MAIL : info@disposafe.com
Web : www.disposafe.com
CUSTOMER CARE NO. +91-129-4170201



CECTREP

**Cepartner4U BV., ESDOORNLAAN
13, 3951 DB MAARN, THE NETHERLANDS.**
E-mail : office@cepartner4u.com

AW/IFU_1337_Rev_00, Date 28-08-2021