

clogged. The air filter last up to 500 hrs of use before replacement . A general rule is to replace the air filter whenever the nebulizer unit is replaced (every 6 months). To replace the air filter, ensure the compressor is turned off, open the air filter cover to take out the air filter and replace it with a new one. Place the cap back into the socket of the compressor and snap the filter cover back in place.

NOTE : Do not attempt to wash and reuse the air filter. Do not use substitute filter material. Compressor damage may result.

CLEANING THE COMPRESSOR HOUSING :

Use a clean, damp cloth to wipe the exterior of the compressor housing.

MAINTENANCE AND REPAIR :

Never operate the nebulizer if it is improperly or incompletely assembled, or damaged. Use only our genuine parts, accessories, tubing and attachments for the nebulizer.

Use of the other parts, accessories, tubing and attachments may damage the nebulizer or cause improper function. Do not open or disassemble the compressor unit at any time. This action will void the warranty. Repairs, service and maintenance should be carried out only us authorized technician.

NOTE : There are no user serviceable parts inside the this nebulizer, Service by anyone other than authorized serviceman voids the warranty.

10. NEBULIZER SPECIFICATIONS

Dimensions

Compressor only)10.7"X 6.6" x 4.5"Weight
(Compressor only)4.0 pound (1.83 kg)
Electrical Requirements120V/60Hz 220V/50Hz
Power Consumption1.4A/68 Watts under normal load
Particle Size0.5µ m to 5 µm Ambient Operation
Temperature Range/Humidity Range/High hydrostatical pressure : 10°C- 40°C (50°F-104°F) / 30% to 75% RH/ 860 to 1060 hpa
Transport/Storage/ Ambient temperature /Humidity High Hydrostatical pressure-25°C-70°C (-13°F-158°F) / 10% to 90% RH / 860 to 1060 hpa Equipment Class & TypClass II equipment

11. LIMITED WARRANTY


We offer a limited one year warranty on the compressor unit. We warrant that the compressor unit and its parts will be free from defects in workmanship and materials for a period of one year from the date of original retail purchase . This warranty does not extend to failures resulting from accident, misuse, abuse, alternation, use of unauthorized service, parts or cleaning solutions, or failure to comply with User's Manual. Accessories are not covered under warranty.

If the warranted equipment should fail during the warranty period, we at our option, will repair or replace part of issue, provided the claim is bona fide it shall be the responsibility of the purchaser to pack and return the equipment in a manner to avoid shipping damage. The shipment must be prepaid to us. This warranty covers the cost of labor incurred in the removal or replacement of the warranted component covers parts at us only.

12. SERVICE AND REPAIR

If you should have any questions about this nebulizer.

1. Please contact us.
2. In writing, describe the malfunction or symptoms of the product. Include your full name, address and daytime contact telephone number, For tubing and accessories, clean , disinfect and sterilize prior to returning Dry the nebulizer unit and place in a plastic bag.
3. Send the written description and the product in a protective box to the address provided by us.
4. Please enclose original purchase invoice & duly filled & stamped warranty card.
5. Please refer to enclosed warranty card.

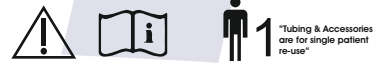

Disposafe Health and Life Care Limited
Plot No.1 &2, Phase-II, Sector-59, Ballabhgarh,
Faridabad-121004, Haryana (INDIA)
E-mail : info@disposafe.com, Web. : www.disposafe.com
Customer Care No. : 91-129-4170201



WARRANTY CARD

Safe-NEB™
Nebulizer **Mini**

MODEL NUMBER	LOT	SERIAL NO.
NEB-MINI		

SALES RECEIPT/INVOICE NO. : _____ DATE _____

CUSTOMER NAME : _____

ADDRESS : _____

CITY : _____ PIN _____

CONTACT NO. : _____

DEALER/RETAILER'S NAME : _____

DEALER SIGN & STAMP

CONDITIONS FOR WARRANTY

1. SAFE -NEB (MINI) has a warranty for the period of one year from the date of purchase against manufacturing defect claimed during the warranty period.
2. If any defect developed by this Nebulizer (MINI) during the warranty period the same will be repaired free of cost.
3. If any defect developed by this Nebulizer (MINI) during the warranty period. please bring it along with warranty card and sales receipt/invoice to the dealer/retailer, from whom you have purchased. In case of any problem, Please send it at your cost (Freight paid) to the company address.

4. Company shall not be responsible for any loss, damage, breakage etc. in transit.
5. For there are no serviceable parts inside the Nebulizer (MINI) during, please do not tamper it.
6. Do not attempt to disassemble or repair the Nebulizer (MINI) by yourself.
7. If the Nebulizer (MINI) sent to the company for repair during the warranty period is found to be under company warranty policy, it shall be returned after repair within 30 days by courier/post at company expenses, else it will be forfeited.
8. Please enclose warranty card (duly filled and stamped by dealer/retailer) and invoice, if warranty card is not duly filled and stamped, warranty claim will not be entertained.

Warranty Does Not Apply to :

- a) Normal wear & tear.
- b) The defect which are caused due to mishandling, damage/breakage during transportation or caused due to natural calamities.
- c) Any repair done by unauthorized person other than the company.
- d) Water seepage inside the main unit.
- e) This warranty does not cover accessories i.e air tube, air filters, mask etc.

Disposafe Health and Life Care Limited
Plot No.1 &2, Phase-II, Sector-59, Ballabhgarh,
Faridabad-121004, Haryana (INDIA)
E-mail : info@disposafe.com, Web. : www.disposafe.com
Customer Care No. : 91-129-4170201