

INSTRUCTION FOR USE :-

- Check the package sealing integrity to ensure sterility.
- Open the yankauer suction device packing to expose the connection port.
- Ensure the yankauer handle is connected to suction tubing and is connected to an appropriate suction device.
- If variable yankauer handle is available, adjust the suction pressure accordingly (Crown/ Plain tip)
- Test for adequate suction by occluding yankauer's side port.
- With the side port remaining opening of yankauer handle, place at operating site.

Caution :

- Do not use if package is open or damaged.
- Do not store in direct sunlight at extreme temperature or in high humidity.
- Discard after use. - Do not re-use. - Do not resterilise