

PRODUCT DERMAL CURETTE

- Size: 2.0mm, 3.0mm, 4.0mm, 5.0mm, 7.0mm
- Sterilized by EO

MATERIALS USED

- Stainless Steel, ABS Plastic PP

INDICATIONS

- For obtaining diagnostic full thickness skin specimen.
- Different size from different skin area of body.
- Abrasive skin tissue.

CONTRAINDICATIONS

- Re-used of blades can work as carrier for communicable disease to patient and/or user.
- Adverse event may be happen if blades are use after expiry date as expiry of product is expiry date of sterility.

INSTRUCTION FOR USE

- Select proper type and size of Dermal Curette as per skin of specimen.
- After selection clean and sight with proper disinfectant.
- Inspect package of Dermal Curette for its intactness expiring.
- With group on handle choose the skin for cutting skin specimen.
- Curette needs to move parallel to upper dermic after reaching the desired layer and depth.
- Remove the Dermal Curette after cutting the desire dept. of skin.
- Take the sample of specimen.

WARNINGS :

- Read instructions for use
- The product should be use only by a qualified surgeon, doctor or a paramedic.
- Before use always check integrity of product and packing along with expiry date.

- For single use only If re-used this can work as carrier for communicable, disease, HIV, Hepatitis, contagious dieses, undue disease to patient and /or user.
- Use product immediately after opening the pack.
- Disposafe is not responsible for any possible consequences resulting for improver use.
- Do not hold any responsibility if device re-used or re-sterile.
- Sterility of product is not guaranteed if packed is broken.
- Re-sterilization and re-use of blade cause to change in mechanical propertied and material used
- Re-sterilization and re-used of blade may not meet the intended use as blade may be blunt.

STORAGE CONDITION

- Keep away from direct sunlight.
- Keep away from rain.
- Store in cool and dry place.

DISPOSAL OF USED DERMAL CURETTE




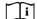














- After use Dermal Curette and pouches shall be disposed off in compliance with the local regulatory requirement of handling medical waste.

PRECAUTIONS

- Always open the pouch from peel a part direction to avoid injury.
- Device are extremely sharp, use care while handling.
- Care must be taken so that the packing is not opened in an unsterile area otherwise blade which has already been sterilized by ETO will become unsterile. Proper procedure must be used as applicable for handling any sterile

product.


- Care must be taken during disposal of device to avoid any contact or injury due to the sharp nature of the device.
- In case of changes in the performance of device for intended use replace the defective device by new to full fill the required application.

 Reference No.	 Medical Device
 Lot No./Batch No.	 Consult Instructions For Use
 Manufacturer	 Do Not Use If Package is Damage and Consult Instructions For Use
 Date Of Manufacturing	 Sterilized By Ethylene Oxide Single Sterile Barrier System
 Use By / Expiry Date	 Authorised Representative In The European Community
 Do Not Reuse	 Temperature Limit
 Do Not Sterilize	 Do not expose to heat or direct sunlight.
 Non-Pyrogenic	 Keep dry
 Caution	
 Moisture limitation	



Disposafe Health and Life Care Limited

Plot No. 1 & 2, Phase-II, Sector-59, Ballabhgarh,
Faridabad -121004, Haryana, INDIA
E-mail : info@disposafe.com
Web : www.disposafe.com
Customer Care No. +91-129-4170201
Mfg. Lie. No.: MFG/MD/2019/000106

 CEpartner4U B.V., Eindhoven 13,
3951 DB Maarn, The Netherlands,
E-mail: office@cepartner4u.com
AW/IFU_1333A REV:03, Dt. 15-04-2024