

# INSTRUCTION FOR USE

## DISPOFLEX THREE WAY STOP COCK (LIPID RESISTANT)

### Material:

Polypropylene, H.D.P.E, Polycarbonate

### Indications:

- Infusion of I.V. Solution.
- Intermittent intravenous drugs administration.
- For administration of high viscous fluids.

### Contraindications:

- Product should not be used in patients with known hypersensitivity
- Not to be used for photo sensitive drugs.
- Not to be used for blood transfusion.

### Instructions for use:

- Check visually whether the package is intact.
- Open packaging and remove stopcock from sterile package.
- Connect one or two I.V. Infusidime as desired to the ports.
- Remove protector from male portion, hold it upright.
- Open clamps of I.V. Infusidime and displace air entirely from
- Stopcock by rotating tap to open position
- Rotate tap to close position.
- Connect it to vein puncture device & secure it with rotation lock.
- Rotate the tap to open position and adjust the flow of infusion
- Re-use of this device may change its mechanical, biological features and may cause device failure, allergic reaction or infections.
- Contains universal 6% taper that can be connected to any standard medical device.

### Warning:

- To be used by a qualified doctor or a paramedic.
- Read instructions before use.
- The product should be used according to the instructions for use.
- **MANUFACTURER DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.**
- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use.
- Improper transport and handling may cause structural and/or functional damage to device or packaging.
- The product is guaranteed non-toxic, sterile and Pyrogen free if the package is not opened or damaged.
- Do not clean or re-sterilize.
- The product should be used immediately after opening the packaging.
- The device must not be re-used after its usage, it can be a potential hazard.

### Storage Condition :

- Do not expose to heat or direct sunlight


**MD** Medical Device


**REF** Reference No.

**LOT** Lot No./Batch No.


 Manufacturer


 Date Of Manufacturing


 Use By / Expiry Date


 Sterilised By Ethylene Oxide Single Sterile Barrier System


 Do Not Reuse


 Do Not Resterilize

 Non-Pyrogenic

 Do Not Use If Package Is Damaged

 Temperature Limit

 Caution

 Consult Instructions For Use

**EC REP** Authorised Representative In The European Community

### Disposa Health and Life Care Limited

Plot No. 1 & 2, Phase-II, Sector- 59, Ballabgarh, Faridabad-121004, Haryana (INDIA)  
E-mail: info@disposafe.com, Web: www.disposafe.com  
CUSTOMER CARE NO. : +91-129-4170201

**EC REP** CPartner4U B.V.  
ESDOORNLAAN 13, 3951 DB MAARN, THE NETHERLANDS  
E-mail : office@cepartner4u.com

  
**0123**