

safe-b.t

Blood Transfusion Set

Material:

-PVC,HDPE,ABS,PP,SS304,Latex

Indications:

- Infusion of I.V. Solutions.
- Intermittent intravenous drug administration.
- To maintain hydration and/or correct dehydration if patient is unable to take sufficient volume of oral fluid.

Contraindications:

- Product should not be used in patients with known hypersensitivity to any of the materials use.
- Administration of high viscous fluids.
- Device must not be used other than as indicated in intended use.

Instructions for use:

- Check the package sealing integrity to ensure sterility.
- Remove the transfusion set using aseptic technique.
- Close Roller Clamp.
- Remove protective cap from closure-piercing device and insert into the outlet of blood container.
- Suspend container, half fill lower portion of drip chamber by squeezing and releasing.
- Do not fill completely.
- Remove protective cap from needle by holding needle upright, open roller clamp and allow blood to displace air entirely from the tubing and the needle. Close roller clamp.
- Perform vein puncture and regulate flow by opening roller clamp.

Warning:

- To be use this product by a qualified doctor or a paramedic.
- Read instructions before use.
- The product should be used according to the instruction for use.

MANUFACTURER DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use.
- Improper transport and handling may cause structural and/or functional damage to device or packaging.
- The product is guaranteed non-toxic, sterile and Pyrogen free if the package is not opened or damaged.
- Do not clean or re-sterilize.
- Do not expose to heat or direct sunlight.
- The product should be used immediately after opening the packaging.

INSTRUCTION FOR USE

REF

Reference No.

LOT

Lot No./Batch No.



Manufacturer



Date of Manufacturing



Use By / Expiry Date



Sterilised By Ethylene Oxide
Single Sterile Barrier system



Do Not Reuse



Do Not Resterilize



Non-Pyrogenic



Do Not Use If Package is Damage



Caution



Temperature Limit



Consult Instructions For Use



Authorised Representative
In The European Community

MD

Medical Device

UDI

Unique Device Identifier

Disposafe Health and Life Care Limited

Plot No. 1 & 2, Phase-II, Sector-59, Ballabhgarh,
Faridabad-121004, Haryana (INDIA)
E-mail : info@disosafe.com, Web : www.disosafe.com
CUSTOMER CARE No. : +91-129-4170201

EC REP

CEpartner4U B.V

ESDOORNLAAN 13, 3951 DB MAARN, THE NETHERLANDS.
E-mail: office@cepartner4u.com

0123