

# DISPOPLUS

## INSTRUCTION FOR USE

### THREE WAY STOP COCK WITH EXTENSION TUBE

#### Material:

Polypropylene, H.D.P.E, Polycarbonate

#### Indications:

- Infusion of I.V. Solution
- Intermittent intravenous Drugs administration

#### Contraindications:

- Not to be used for high viscous fluids and lipids administration.
- Product should not be used in patients with known hypersensitivity to any of the materials used.
- Not to be used for blood transfusion.

#### Instructions for use:

- Peel open the pouch.
- Unscrew the protective cap from female luer.
- Connect male portion of device with female luer of tubing.
- Remove protective cap from male luer side 3 way stop cock.
- Hold Three way stop cock upright and open clamp of device and Three way stop cock allow the solution to displace air entirely from tubing.
- Connect Three way stop cock veinpuncture device.
- Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or infections.

#### Warning:

- To be use this product by a qualified doctor or a paramedic.
- Read instructions before use.
- The product should be used according to the instruction for use.

#### **MANUFACTURER DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.**

- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use.
- Improper transport and handling may cause structural and /or functional damage to device or packaging.
- The product is guaranteed non-toxic, sterile and Pyrogen free if the package is not opened or damaged.
- Do not clean or re-sterilize.
- Do not expose to heat or direct sunlight.
- The product should be used immediately after opening the packaging.

REF

Reference No.

LOT

Lot No./Batch No.



Manufacturer



Date of Manufacturing



Use By / Expiry Date



Sterilized By Ethylene Oxide  
Single Sterile Barrier System



Do Not Reuse



Do Not Resterilize



Non-Pyrogenic



Do Not Use If Package is Damaged



Temperature Limit



Caution



Consult Instructions For Use

EC REP

Authorised Representative  
In The European Community

MD

Medical Device

**Disposafe Health and Life Care Limited**  
Plot No. 1& 2, Phase-II, Sector-59, Ballabhgarh,  
Faridabad-121004, Haryana ( INDIA)  
E-mail : info@disposafe.com  
Web : www.disposafe.com  
CUSTOMER CARE No. : 91-129-4170201

EC REP

CEpartner4U B.V., ESDOORNLAAN 13,  
3951 DB MAARN, THE NETHERLANDS.  
E-mail: office@cepartner4u.com  
AW/IFU\_552, REV-04, Dt. 25-03-2023

**CE** 0123