

POWDERED/POWDER FREE STERILE LATEX SURGICAL GLOVES

Material: Natural Rubber Latex/NITRILE

Colour : Creamy White

Shape : Anatomic

Cuff : Beaded, External

Surface : Textured

Internal Surface : Powdered/Powder free Size : 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0 Surgical gloves are used as a barrier to prevent the transmission of the infection between health care professionals and patients. Surgical gloves are of higher quality than normal gloves.

DEVICE INTENDED USE:

- These disposable gloves is made up of natural rubber latex which is anatomically shaped with thumb position towards the Palm side of the index finger which reduces the fatigue on the hands, intended to be worn on the hands usually in surgical settings to provide barrier against potentially infectious fluids and other contaminants.

Storage Condition :

- Gloves must be stored in cool dry environment which is dust free. Cartons and Boxes must be stored unopened until required. Recommended storage temperature is 10°C-45°C. Avoid exposure to direct light, heat and excessive humidity. As ozone is deleterious, storage area should not contain any equipment capable of generating ozone such as ultraviolet light, fluorescent lights, mercury vapour lamps, photocopier, high voltage equipment, X-ray units, electric motors and electro surgical equipments.

INSTRUCTION FOR USE :

Directions to wear the surgical gloves

- Check the packing sealing integrity to ensure sterility.
- Must check the date of manufacturing and expiry date before use.
- Select the appropriate size Gloves for the hand, Take out the Gloves from the pouch.
- Wear the gloves as left to the Left hand & Right to the Right Hand as denoted on the wallet.
- Adjust the gloves to cover wrists as required.

INSTRUCTION FOR USE

Directions to remove the surgical gloves

- Grasp glove on the outside about 1/2 inches below the cuff (edge of the glove opening) Do not touch the wrist with the other hand.
- Pull down glove, turning it inside out. Hold the inside-out glove in the glove hand.
- Gather the inside-out glove in the gloved hand.
- Insert finger of the bare hand under the cuff of the gloved hand.
- Pull down the glove until it is inside out, drawing it over the first glove.
- Discard gloves in a garbage container (This step reduces the spread of microorganisms).

CONTRAINDICATIONS :

- Latex gloves are made of Natural rubber latex, which may cause allergic reactions including Latex Allergy Anaphylactic if the user is allergic to latex.
- Gloves contain Natural Latex; persons who are sensitive to Latex should consult a physician before using.

INDICATIONS:

- Protection of the user from contamination with blood, body fluids and excretions.
- Prevention of pathogen release from the hand which may cause infection into the surgical site.

WARNING AND PRECAUTIONS :

- Do not use after the expiry date printed on the label.
- Do not use if the Package is opened or damaged.
- Do not Re Use - The reuse of the gloves may cause severe complications such as cross contamination, dermatitis, loss of physical properties, infection, allergic reaction and poor barrier protection.
- Latex gloves are made of Natural rubber latex, which may cause allergic reactions including Latex Allergy Anaphylactic if the user is allergic to latex.
- Single Use Packaging label and Sales Packaging Label.
- Device should not be stored in direct sunlight.

REF

Reference No.

LOT

Lot No./Batch No.



Manufacturer



Date of Manufacturing



Use By / Expiry Date



Sterilized By Ethylene Oxide
Single Sterile Barrier System



Do Not Reuse



Do Not Resterilize



Non-Pyrogenic



Do Not Use If Package is Damaged
and Consult Instructions For Use



Temperature Limit



Caution



Consult Instructions For Use

EC REF

Authorised Representative
In The European Community

MD

Medical Device

CE 0123

Disosafe Health and Life Care Limited

Plot No. 1& 2, Phase-II, Sector-59, Ballabhgarh,
Faridabad-121004, Haryana (INDIA)
E-mail : info@disosafe.com
Web : www.disosafe.com
CUSTOMER CARE No. : 91-129-4170201

EC REP

CEpartner4U B.V., ESDOORNLAAN 13,
3951 DB MAARN, THE NETHERLANDS.

E-mail: office@cepartner4u.com

AW/IFU_460, REV:01, Dt. 28-11-2023