

DISPOSCALP Scalp Vein set

MATERIAL USED :

- PVC, SS, P.P.,

INDICATIONS :

- Infusion of I.V. Solutions
- Routine blood collection Procedure

CONTRAINDICATIONS:

- Product should not to be used in patients with known hypersensitivity to any of the materials used.
- Administration of high viscous fluids.
- Large blood transfusion.

INSTRUCTION FOR USE :

- Carefully select and aseptically prepare the site.
- Select suitable size of inspect visually to ascertainment that package is intact.
- Remove Needle Cover.
- Grip the S.V. Set wing and insert in vein & check if blood is come in the tube.
- Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions of infections.
- Contains universal 6% taper that can be connected to any standard medical device.
- The needle is coated with silicon oil any person who is allergic to silicon oil must consult practitioner before use of this product.

DURATION OF USED : 72 Hours

WARNING :

- Do not expose to heat or direct sunlight.

INSTRUCTION FOR USE

- The use of this product is restricted to a qualified doctor or a Paramedic
- Read Instruction before use.
- After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and country laws and regulations.

MANUFACTURER DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use. Improper transport and handling may cause structural and/or functional damage to device or packaging.
- The product is guaranteed sterile & Non-pyrogenic if the package has not been opened or damaged.
- Do not clean or re-sterilise. For single use only. Discard after use.
- The product should be used immediately after opening the packaging.
- The device must not be re-used after its usage, it can be a potential biohazard.

Disposafe Health and Life Care Limited

Plot No.: 1 & 2, Phase-II, Sector-59, Ballagbarh,
Faridabad -121004, Haryana, INDIA

E-mail : info@disposafe.com

Web : www.disposafe.com

Customer Care No. 91-129-4170201

Mfg. Lic. No. MFG/MD/2019/000106

 0123

 EC REP

CEpartner4U B.V.,
ESDOORNLAAN 13,
3951 DB MAARN,
THE NETHERLANDS,

E-mail: office@cepartner4u.com



Medical Device



Reference No.



Lot No./Batch No.



Manufacturer



Date Of Manufacturing



Use By / Expiry Date



Sterilised By Ethylene Oxide



Do Not Reuse



Do Not Resterilize



Non-Pyrogenic



Do Not Use If Package Is Damaged



Caution



Consult Instructions For Use



Temperature Liimit



Authorised Representative
In The European Community

AW/IFU_1441, REV 02, DT.: 03-05-2023