

----- TEAR HERE -----



We Touch.... Lives!

# nasal-safe™

## TWIN BORE NASAL OXYGEN SET

With 2 mtr. Long Multichannel kink Resistant Tube

### Instruction For Use :

- ▶ Attach the set to the Patient's face by inserting Cannula tips into the nostril.
- ▶ pass tubing over both ears and position under the chin. Slide adapter up tubing to adjust for comfort and fit.
- ▶ Tighten only enough to secure tube in nostrils.
- ▶ Attach the other end of oxygen cannula to the oxygen outlet. Adjust flow as directed by physician.

### Caution :

- ▶ Single patient use only.
- ▶ Do Not Store In Direct Sunlight And Extreme Temperature Or In High Humidity.
- ▶ Do not use if package is open or damaged.

### Disposafe Health and Life Care Limited

Plot No. 1 & 2, Phase-II, Sector-59, Ballabhgarh  
Faridabad-121004, Haryana, INDIA  
E-mail : info@disposafe.com,  
Web : www.disposafe.com  
Customer Care No. 91-129-4170201  
Mfg. Lic. No.: MFG/MD/2019/000106

<b>05</b> Disponibles <b>Caution :</b>	<ul style="list-style-type: none"><li>• Non Toxic</li><li>• Discard after single use.</li><li>• Use only if the package is intact.</li></ul>
<b>07</b> Monouso <b>Attenzione</b>	<ul style="list-style-type: none"><li>• Non Tossico</li><li>• Gettare dopo l'uso.</li><li>• Usare solo se la confezione e'Integra.</li></ul>
<b>08</b> Jetables <b>Attention :</b>	<ul style="list-style-type: none"><li>• non Toxique</li><li>• Jeter après usage.</li><li>• Ne pas utiliser si l'emballage est endommagé</li></ul>
<b>06</b> Materiales desechables• No Tóxico <b>Atención :</b>	<ul style="list-style-type: none"><li>• desecar desés pude un solo uso.</li><li>• Utilizar sólo si el paquete está intacto.</li></ul>
<b>04</b> Wegwerproducten • niet giftig <b>Waarschuwing :</b>	<p>Na gebruik wegwerpen. Alleen te gebruiken bij inacte verpakking.</p>
<b>02</b> Descartáveis <b>Cuidado</b>	<ul style="list-style-type: none"><li>• Atóxico</li><li>• Jogue fora após uma única utilização.</li><li>• Use apenas se a embalagem estiver intacta.</li></ul>
<b>07</b> Descartavel <b>Catutela</b>	<ul style="list-style-type: none"><li>• Intoxicavel</li><li>• Descare acos uso.</li><li>• Usar somente quando o Pacote estiver intato</li></ul>

**EC REP** CEpartner4U BV., ESDOORNLAAN 13,  
3951 DB MAARN, THE NETHERLANDS.  
E-mail : office@cepartner4u.com

**SIZE :**

**REF :**

**LOT :**

**MD :**

**Unit :**



AW/IFU\_465, REV: 01